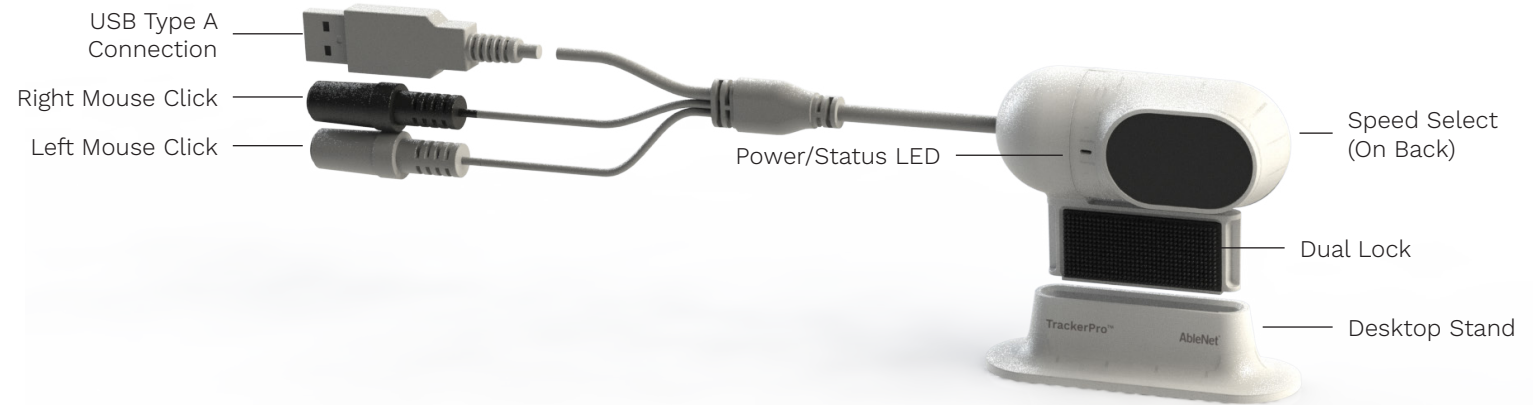
AbleNet[®]

TrackerPro™ 2

www.ablenetinc.com
(800) 322-0956

Getting Started:

1. Place TrackerPro 2 directly in front of the user. TrackerPro 2 can be attached to a monitor using the included dual lock or placed on a flat surface using the included desktop stand.
2. Plug TrackerPro 2 into available USB Type A port on computer. The Power/Status LED on TrackerPro 2 will turn red.
3. Place a TrackerPro 2 reflective dot on forehead of user.
4. Position user 12-in to 36-in (30-cm to 91-cm) away from TrackerPro 2.
5. The Power/Status LED on TrackerPro 2 will turn green once it detects the reflective dot on the user.
6. Small head movements will move the mouse cursor on the computer. If the user is unable to move the mouse cursor across the entire screen, try adjusting the mouse cursor speed.

Mouse Click Options:

- Plug one or two switches into TrackerPro 2. The white switch jack is for left mouse click and the black switch jack is for right mouse click.
- For use without switches, use a dwell selection software.
 - Windows: Dwell Clicker 2 (purchased separate)
 - macOS Mojave or newer: Dwell selection is part of the accessibility settings for the keyboard
 - Google Chrome OS: Dwell selection is part of the accessibility settings

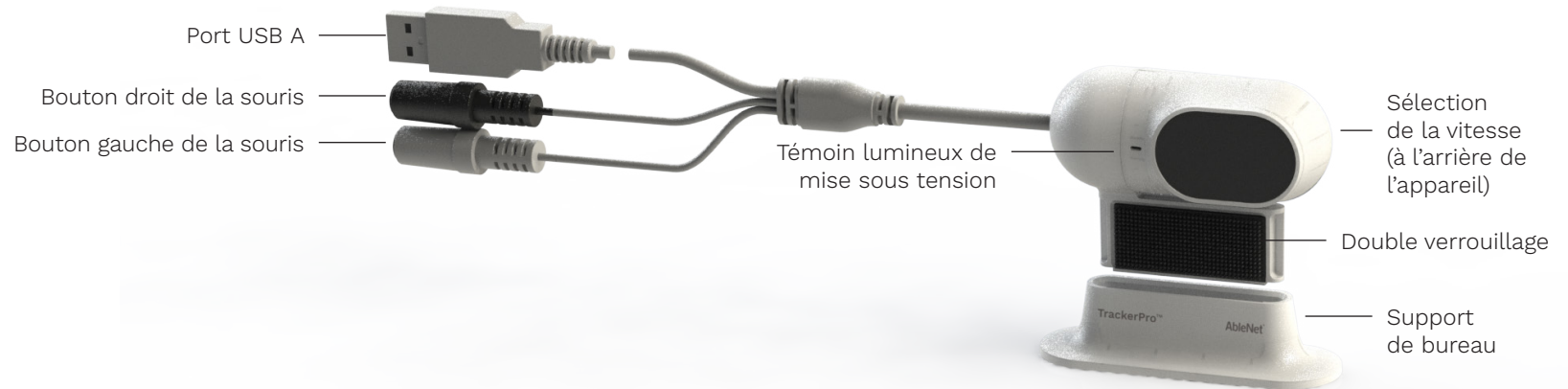
Mouse Cursor Speed Adjustment:

- On the back of TrackerPro 2, move the speed selector switch to change mouse cursor speed.
 - I = slow
 - II = fast
- For more control, find the mouse cursor speed adjustment within the computer operating system

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Warranty & Contact Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at www.ablenetinc.com.

AbleNet, Inc. 2625 Patton Road, Roseville, MN 55113
(800) 322-0956 / www.ablenetinc.com

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FCC

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between equipment and receiver.
- Consult AbleNet Technical Support or your local Distributor for help.

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