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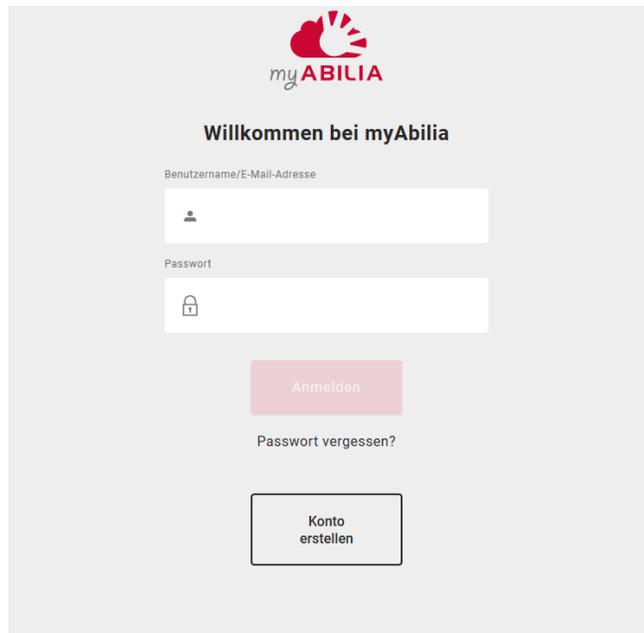
www.myabilia.com

The desktop login page features the myABILIA logo at the top center. Below it, the heading "Willkommen bei myAbilia" is displayed. The login form consists of two input fields: "Benutzername/E-Mail-Adresse" and "Passwort", each with a corresponding icon (person and lock). A pink "Anmelden" button is positioned below the password field. A link for "Passwort vergessen?" is located below the button. At the bottom, a white button with a black border is labeled "Konto erstellen".

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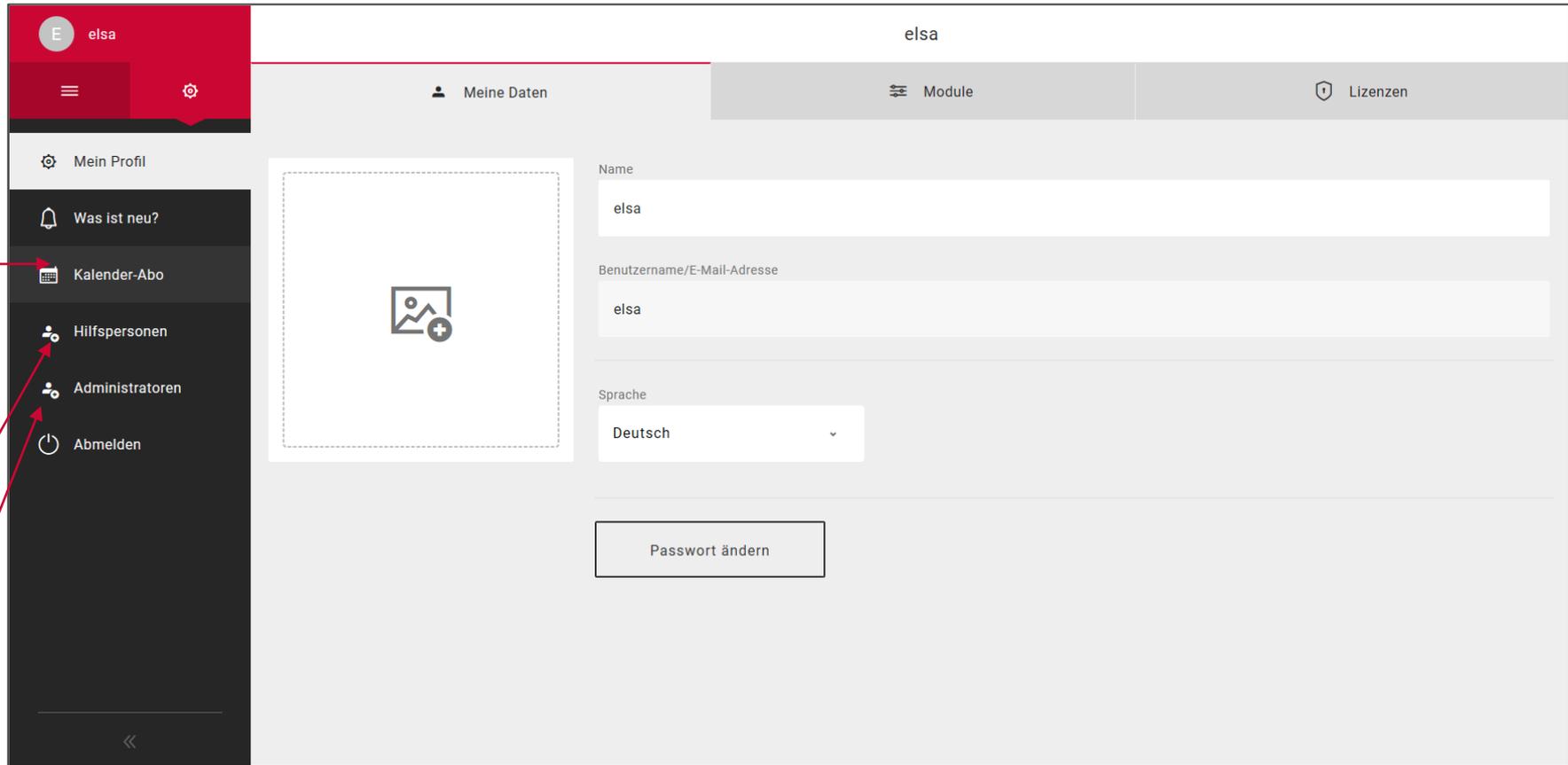
The screenshot shows the 'Neue Aktivität' (New Activity) screen in the ABILIA app. The screen is divided into four tabs: 'Name & Bild', 'Alarm & Erinnerungen', 'Intervall', and 'Verbundene Funktionen'. The 'Intervall' tab is currently selected. Below the tabs, there are three input fields: 'Startzeit' (Start Time) with a clock icon, 'Endezeit' (End Time) with a clock icon, and 'Intervall' (Interval) with a dropdown menu. The 'Startzeit' field contains '09:00', the 'Endezeit' field contains '--:--', and the 'Intervall' dropdown menu is set to 'Einmal'. At the bottom of the screen, there are two buttons: 'Abbrechen' (Cancel) on the left and 'Erstellen' (Create) on the right.

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The screenshot displays the MEMOplanner app interface. At the top, there is a red header with the user's name 'elsa' and a 'Neu' dropdown menu. Below the header, a row of standard activities is shown with icons and labels: 'aufwachen' (alarm clock), 'Gehe ins Badezimmer' (toilet), 'Dusche' (shower), 'Mittagessen' (plate of food), 'Teepause' (cup of tea), 'Einkaufen' (shopping cart), and 'Aufladen MEMOplanner' (phone with plug). A 'Standardaktivitäten' menu is open, showing a list of these activities with radio buttons for selection. The menu title is '+ Neue Aktivität Standardaktivität auswählen'. At the bottom of the menu, there are buttons for '< Vorherige' and 'Nächste >'. A red arrow points from the text 'Erstellen Sie hier Ihre eigenen Standardaktivitäten' to the 'Standardaktivitäten' menu item in the left sidebar.

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