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
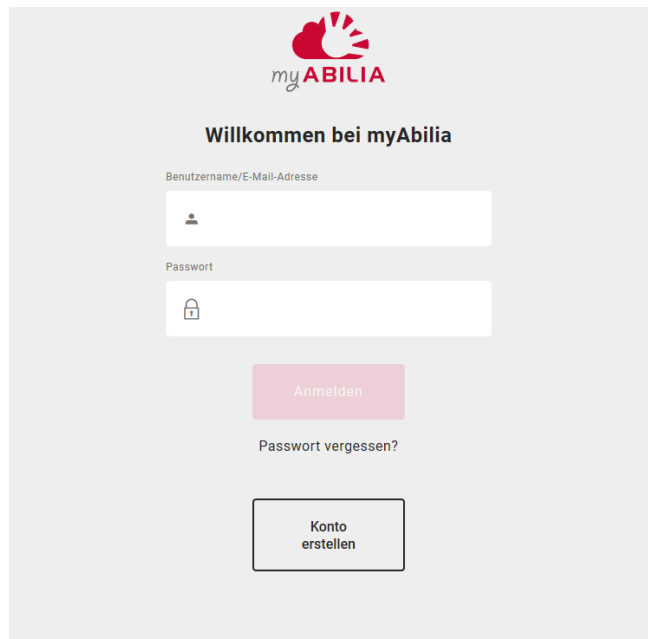
www.myabilia.com

The desktop login page features the myABILIA logo at the top center. Below it, the heading "Willkommen bei myAbilia" is displayed. The login form consists of two input fields: "Benutzername/E-Mail-Adresse" and "Passwort", each with a corresponding icon (person and lock). A pink "Anmelden" button is positioned below the password field. A link for "Passwort vergessen?" is located below the button. At the bottom, a white button with a black border is labeled "Konto erstellen".

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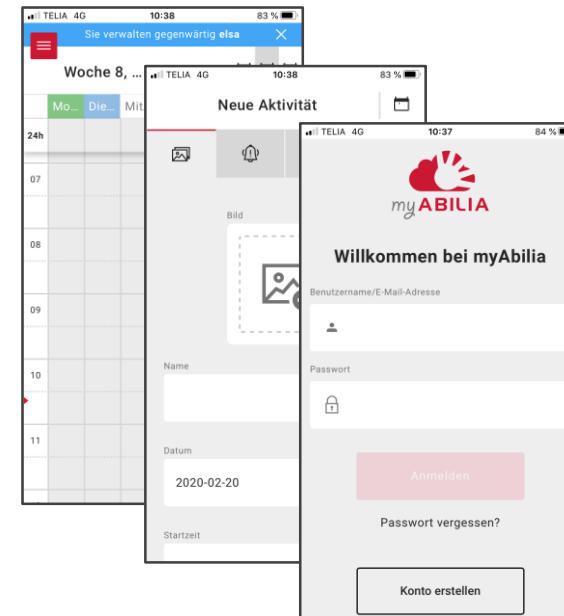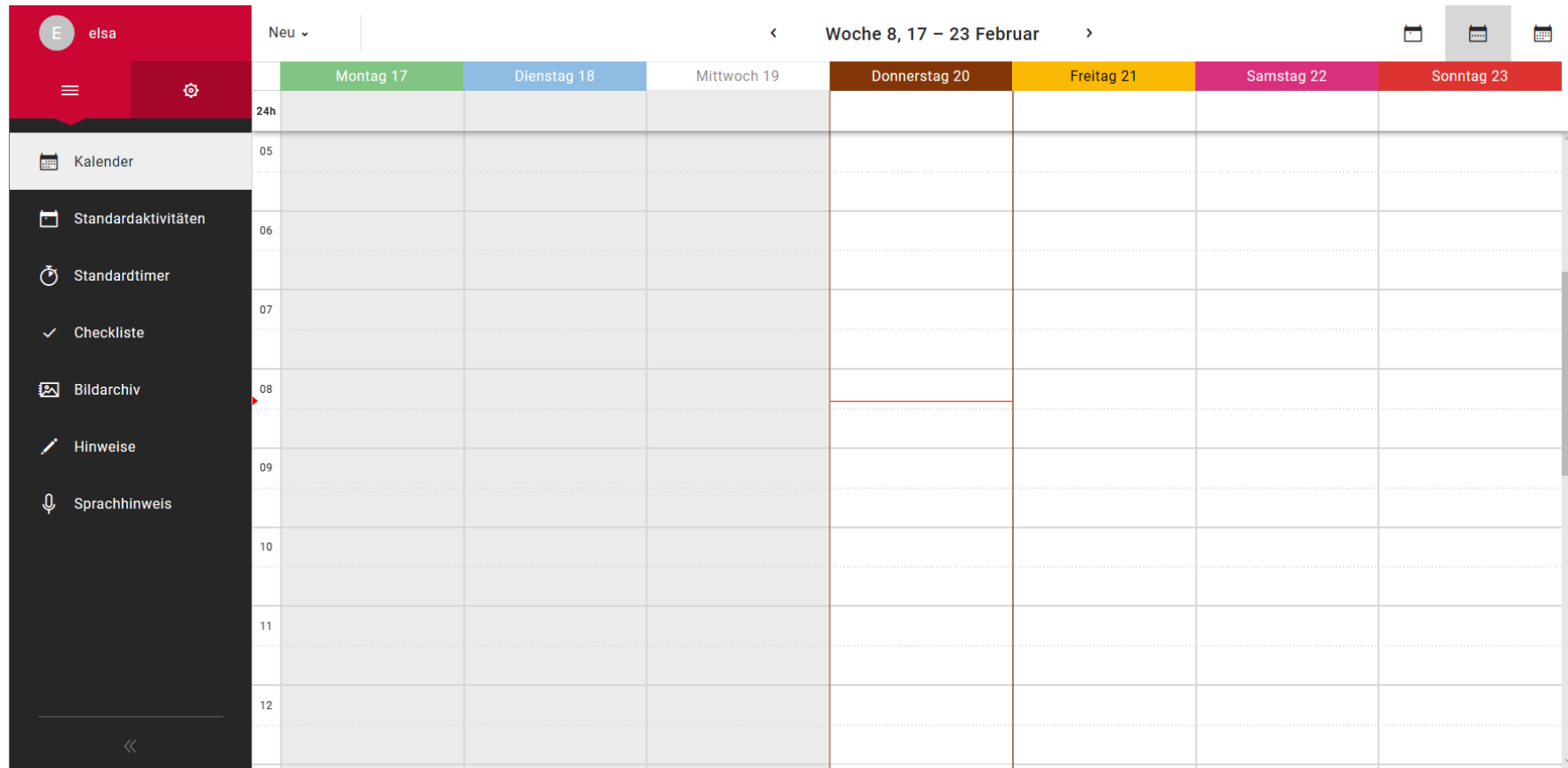
The screenshot displays the ABILIA mobile application interface. At the top, there is a red header bar containing the user's name 'elsa', a menu icon, and a settings icon. Below the header, the main content area shows a weekly calendar overview for the week of February 17-23. The days are color-coded: Montag 17 (green), Dienstag 18 (blue), Mittwoch 19 (grey), Donnerstag 20 (brown), Freitag 21 (yellow), Samstag 22 (pink), and Sonntag 23 (red). The time axis on the left ranges from 24h to 12. A red horizontal line is visible on Thursday, February 20th, at the 08:00 mark. The bottom of the screen shows a back arrow.

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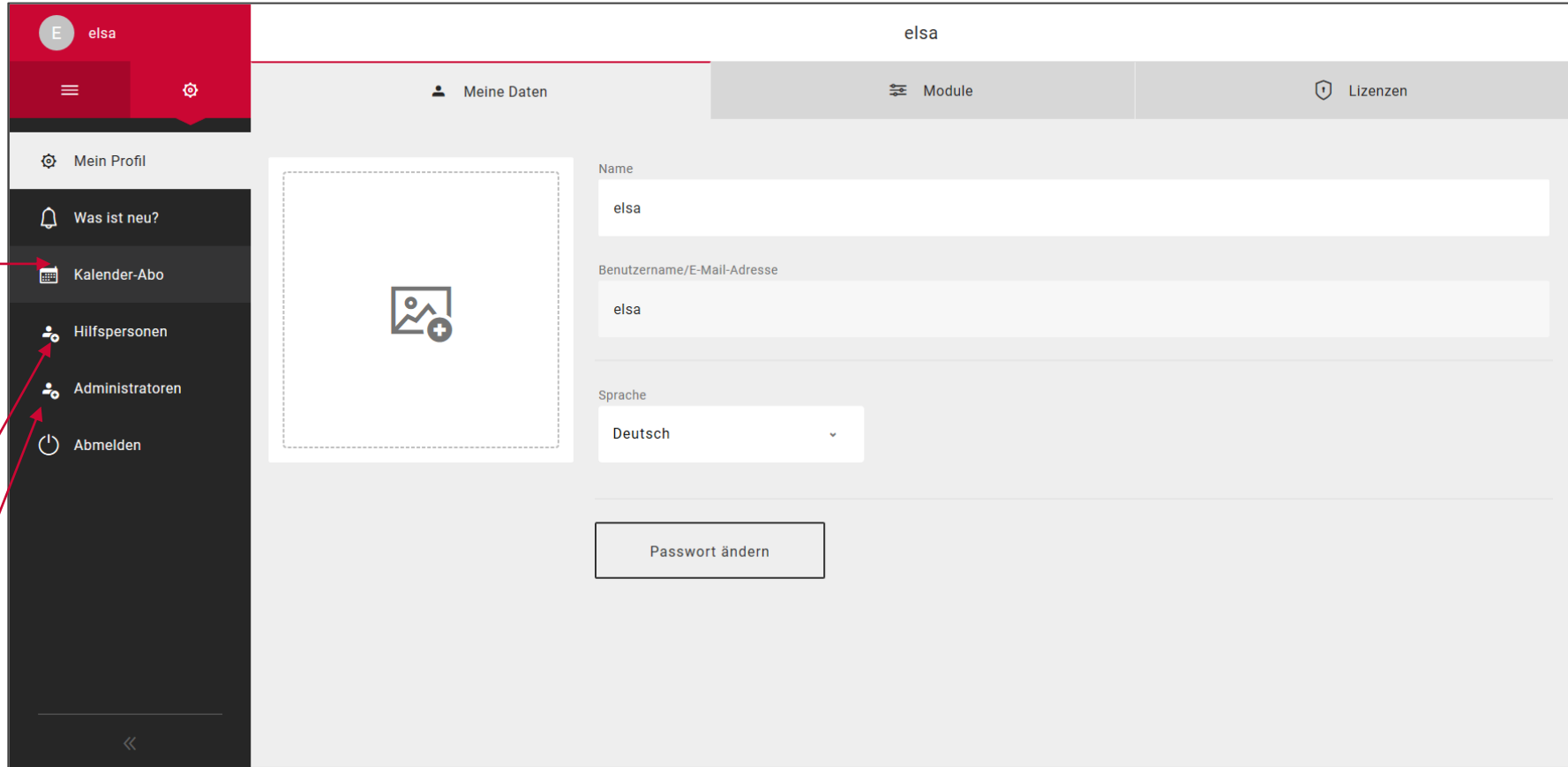
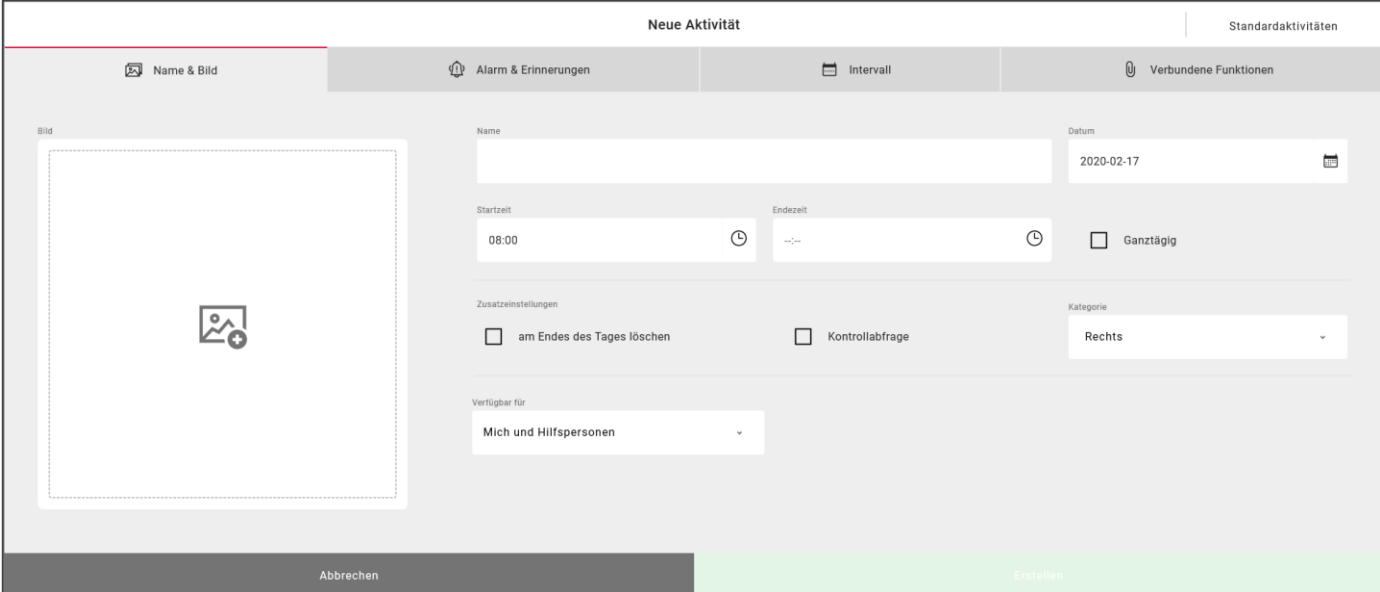
The screenshot shows the 'Neue Aktivität' (New Activity) screen in the ABILIA app. The 'Intervall' (Interval) tab is selected. The screen displays the following fields:

- Startzeit** (Start Time): 09:00
- Endezeit** (End Time): --:--
- Intervall** (Interval): Einmal (Once)

The bottom navigation bar shows two buttons: 'Abbrechen' (Cancel) and 'Erstellen' (Create).

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The screenshot displays the MEMOplanner app interface. At the top, there is a red header with the user's name 'elsa' and a settings icon. Below the header, a navigation menu on the left lists various app features: Kalender, Standardaktivitäten, Standardtimer, Checkliste, Bildarchiv, Hinweise, and Sprachhinweis. The main area shows a grid of standard activities with icons and labels: 'aufwachen' (alarm clock), 'Gehe ins Badezimmer' (toilet), 'Dusche' (shower), 'Mittagessen' (plate of food), 'Teepause' (cup of tea), 'Einkaufen' (shopping cart), and 'Aufladen MEMOplanner' (phone with charging cable). A 'Neu' dropdown menu is visible above the grid. A modal window titled 'Neue Aktivität Standardaktivität auswählen' is open, showing a list of the same activities with radio buttons for selection. At the bottom of the modal, there are buttons for '< Vorherige' and 'Nächste >'. The ABILIA logo is in the bottom right corner.

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